


Quick Start Guide

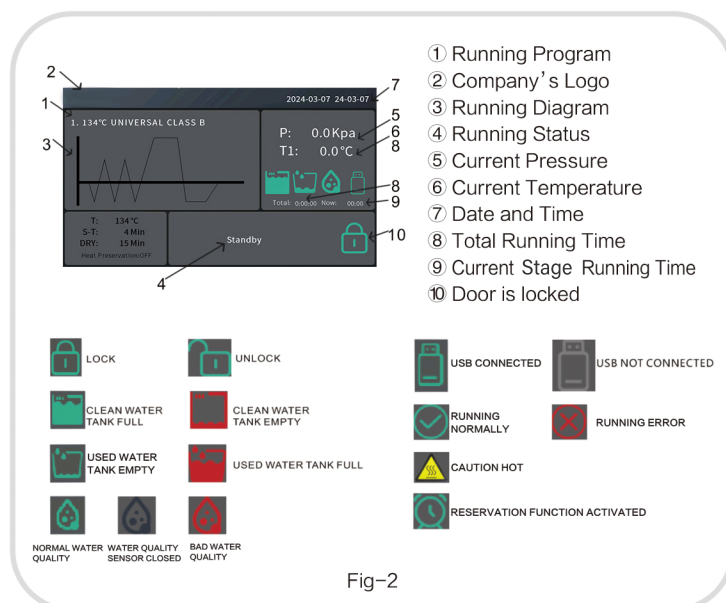

1. The steam sterilizer should be installed in a leveled and drafty counter place allowing minimum 10 cm space at all sides and 20 cm at the top.
2. Please adjust the height of the front foot pad so that the water in the chamber can flow toward the water drainage vent to ensure the drying effect and avoid wet pouches.
3. Please confirm the voltage and the type of plug used in your area in advance and use the power cable come with the machine.
4. After connecting the power supply, turn on the air-circuit breaker, and then turn on the power switch on the upper right side of the machine.
5. Please follow the instructions to open the lid and add distilled water. When lack of clean water, the icon will turn red, please add distilled water in time. If the water quality icon turns red, please clean the clean water tank and add distilled water.
6. Open the door and check the value of P (Fig-2 ⑤ the current pressure) is under 10 Kpa. If it is more than 10 Kpa, please do the pressure correction or contact your supplier.
7. Please place the instruments that need to be sterilized into the chamber as required. Closed the door firmly and check the door icon (Fig-2 ⑩), be sure the door is locked.

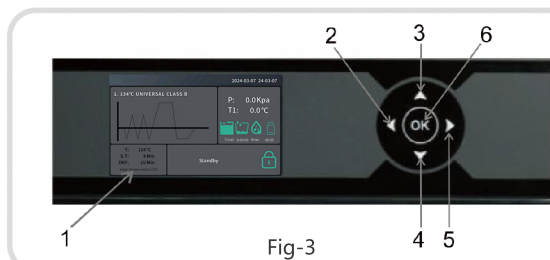


Fig-3

- ① Smart display
- ② Escape button/ Main menu key
- ③ Up selection button
- ④ Down selection button
- ⑤ Confirm button
- ⑥ Start button



Fig-4

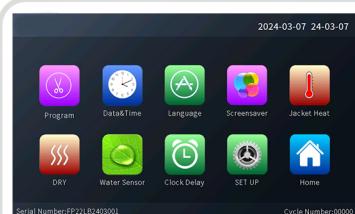


Fig-5



Fig-6



Fig-7

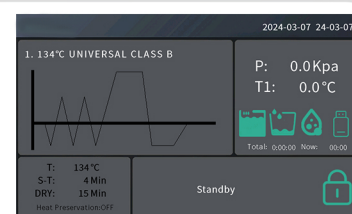


Fig-8

8. Press the ◀ button (Fig-3 ②) or press the 'MENU' icon as Fig-4 to enter the 'MENU'; press the buttons or press the 'Program' icon as Fig-5 to enter the 'Program'; use the ▲ or ▼ buttons or slide cycle icons to select a cycle; press ▶ button (Fig-3 ⑤) or press the selected cycle icon to view the cycle details as Fig-7; press the 'OK' button or press ✓ to select the program and enter the HOME.
9. Please long-press the ▶ button (Fig-3 ⑤) or press the screen to choose 'START' icon as Fig-4 to enter the 'Start?' interface; then press the 'OK' button or press 'OK' to run the cycles. ⚠ Caution: Don't open the door during running!
10. Please long-press ◀ button (Fig-3 ②) or press the 'HISTORY' icon as Fig-4 to enter the history, then you can open, print or copy the sterilization results.